HANDBIKE

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Short Manual

Inhalt

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Content

Safety information	1	Riding your Handbike	7
Read Instruction Manual.....	1	Mounting and dismounting.....	7
Intended use.....	1	Riding	7
Before your first ride.....	1	Shifting gears with the twist shifter	8
Operating the hand crank.....	1	Shifting gears with the Mountain Drive gear system.....	8
Strapping your legs into the footrests	1	Using the Handbike as a tandem.....	8
Adjusting your Handbike	2	Replacement parts	9
Checking the seat position	2	Hand-crank assembly, complete.....	9
Adjusting the frame length and steering	3	Footrests.....	10
Adjusting the footrests	4		
Adjusting the seat angle.....	5		
Adjusting the angle of the steering/crank column ..	5		
Adjusting the length of the steering/crank column	5		
Adjusting the steering sensitivity.....	6		

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Safety information

Read Instruction Manual

Please read this Instruction Manual carefully and familiarize yourself with the construction and function of the vehicle in practice. The Handbike is built on the basis of the Kettwiesel. Therefore, we strongly recommend that all Handbike users read the Instruction Manual for the Kettwiesel, above all the section “Safety information”.

This Instruction Manual is a supplement to the Kettwiesel Instruction Manual and only covers the special features associated with the hand-drive system.

Intended use

The Handbike is a hand-powered recumbent trike designed for use on paved roads and paths. The trike should not be operated in traffic until it has been equipped with the accessories required by the applicable traffic regulations of the country in which it is being used.

Before your first ride...

Your authorized retailer has assembled your Handbike and adjusted it to your specific size requirements. If you need to make adjustments for another rider, please read the section “Adjusting your Handbike” on pages 2 through 6 of this manual. Before your first ride, please take some time to familiarize yourself with the Handbike.

Operating the hand crank

Please be advised that the use of the hand-crank/steering assembly requires the mastery of special movement sequences. Therefore, you should not ride the Handbike in traffic until you feel confident with all aspects of its operation.

Strapping your legs into the footrests

Be sure to use the special straps to fasten your legs to the footrests. This provides you with a secure sitting position and prevents your body from slipping in the seat when braking and cornering.

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Adjusting your Handbike

The proper adjustment of the Handbike is important for riding comfort and performance. Your Hase retailer has adjusted your Handbike to optimally fit your specific size requirements. If major adjustments become necessary, consult your retailer or perform the adjustments according to the following instructions. An additional person may be required for certain procedures. The Handbike offers several adjustment features that can be used in combination to adjust the trike to your specific needs. Please read and observe the following recommendations.

Caution!

Improperly performed adjustments can result in accidents. We recommend all adjustments be performed by a professional bicycle mechanic.

Checking the seat position

In order to create an ergonomic sitting position, the footrests should be adjusted in such a way that the legs are slightly bent and the soles of the feet are resting against the curved section of tubing that forms the stirrup of the footrests. This position is generally the most comfortable. Furthermore, the slight pressure of the feet against the stirrups provides more stability in the seat.

The crankset should be positioned as low as possible. A low position is especially ergonomic. This position also provides the rider with the best field of vision. Please ensure, however, that the crankset is not positioned so low that the handles and/or crank column come in contact with the legs when steering and pedaling. When the handles of the crank arms are positioned farthest from the body, the arms should be slightly bent.

Note:

The standard length for the hand crank arms is 174 mm (6¾ in). This length is suitable for riders whose arms are at least 50 cm (19¾ in) in length. Special 150mm cranks (6 in) are available for smaller riders (accessories).

The recommended procedures for mounting and dismounting the Handbike are described on page 7.

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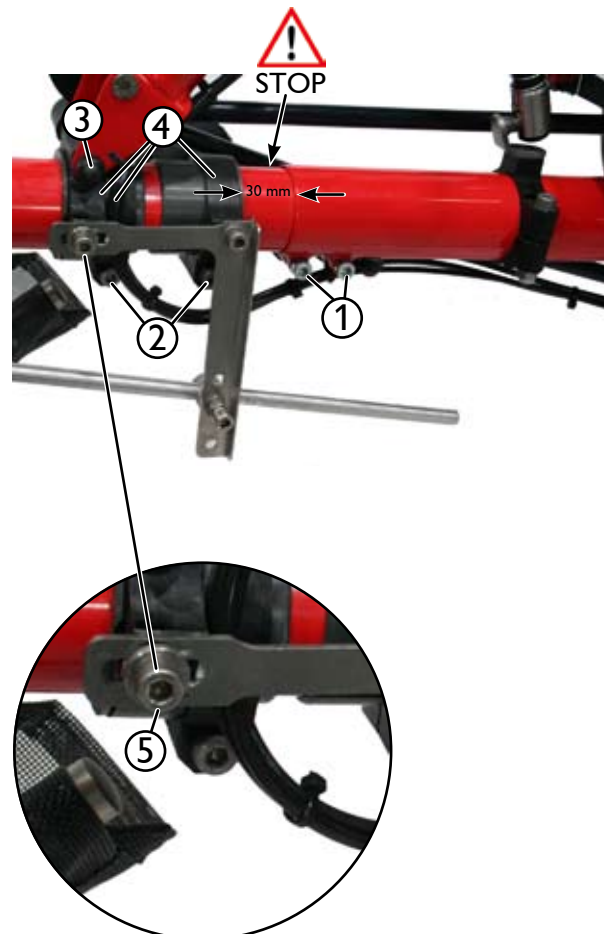
Adjusting the frame length and the steering

Any change in the frame length will also affect the angle of
the steering/crank column because the clamp on which the
column is mounted slides into a new position when the front
boom is adjusted. After the front boom has been reposi-
tioned, the clamp that holds the steering/crank column must
be slid back to its original position (by the same distance the
front boom was moved) in order to restore the desired angle.
Otherwise the chain length must be adjusted. After the frame
length has been adjusted, a readjustment of the steering as-
sembly is required.

Caution!

This work should only be performed by a professional
bicycle mechanic.

1. Loosen the two Allen bolts (1) between the front boom
and the main frame using a 6mm Allen key. These bolts
are located on the left side of the frame (as seen when
facing in the direction of travel).
2. Loosen the bolt (2) on the rear clamp of the steering/
crank column using a 6mm Allen key.
3. Adjust the length of the frame by sliding the front boom
into or out of the main frame tube. Then tighten the bolts
(1) lightly.

Caution!

The front boom must not be extended past the minimum
insertion mark, which is a small hole located on the top
of the boom. This hole should not be visible. If the hole
is visible, the front boom has been extended past its
minimum insertion length. Riding with an over-extended
boom could cause accidents or damage to the frame.

4. Slide the three clamps (4) of the steering/crank column
back along the front boom by the same distance (in a
direction opposite to that in which the front boom was
moved). If the front clamp of the steering/crank column
is too tight to be moved, loosen the bolt (3) on the front
clamp using a 6mm Allen key.
5. Position the rear clamp at a distance of 30 mm (1 3/16 in)
to the main frame. Otherwise the chain length must be
adjusted. Position the two front clamps in such a way that
the bolt (5) that slides in the front slot of the steering rod

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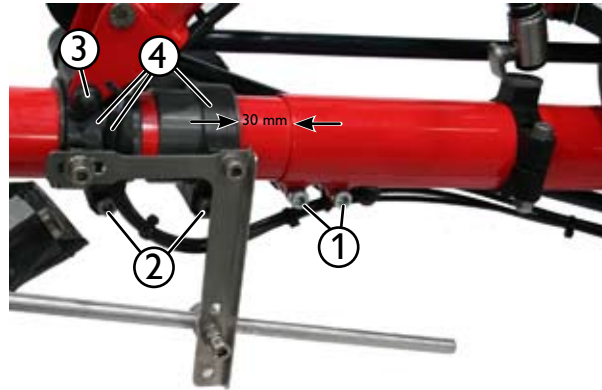
bracket is located in the rear part of the slot when the steering/crank column is upright (not tilted) and the front wheel is aligned with the frame (facing straight ahead).

6. Using a 6mm Allen key, tighten the bolt (3) on the front clamp lightly, ensuring that the steering/crank column can still be easily tilted to the side. Tighten the bolt (2) on the rear clamp using a 6mm Allen key.
7. Tighten the two Allen bolts (1) between the front boom and the main frame using a 6mm Allen key and a torque of 12 Nm (8.9 ft-lbs).

Caution!

It is absolutely necessary to tighten these bolts with a torque of 12 Nm (8.9 ft-lbs). Riding with loose bolts poses a high risk of accident and frame damage.

8. Check that the steering system is functioning properly. When the steering/crank column is upright (not tilted), the front wheel must be facing forward (straight ahead). The steering/crank column must move freely to the left and right, and the front wheel must respond by turning to the left and right, respectively. Ensure that the steering system is not blocked.
9. Check the seat position. The footrests may need to be readjusted.

Adjusting the footrests

1. To change the angle and/or height of the footrest, loosen the bolt (1) on the footrest cylinder using a 6mm Allen key.
2. To adjust the length of the footrests, loosen the two bolts (2) on the footrest tube using a 5mm Allen key.
3. Retighten all bolts.

Caution!

Check to make sure that the footrests are not blocking the steering assembly. Risk of accident!

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Adjusting the seat angle

1. Loosen the two quick releases (4) of the seat stays.
2. Adjust the seat to the desired angle.
3. Retighten the two quick releases (4).

Adjusting the angle of the steering/crank column

Follow these instructions to adjust the horizontal and vertical position of the crank assembly to your arm length and shoulder height.

1. Turn the plastic bolt (1) counterclockwise to lower the steering/crank column or clockwise to raise the column.

Note:

It is also possible to change the angle of the steering/crank column by repositioning the support bar (3).

Adjusting the length of the steering/crank column

If, after adjusting the column angle, the position of the crank assembly is still not optimal, the length of the column can also be adjusted. An optimal position can usually be achieved through a combination of these two adjustments. Any larger adjustment to the length of the steering/crank column requires chain-length adjustment.

1. Loosen the bolt (2) using a 6mm Allen key. Lengthen or shorten the steering/crank column by repositioning the front boom of the column.

Caution!

The front boom of the steering/crank column must not be extended past the minimum insertion mark, which is a small hole located on the top of the boom. This hole should not be visible. If the hole is visible, the front boom has been extended past its minimum insertion length. Riding with an over-extended boom could cause accidents or damage to the frame.

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Adjusting the steering sensitivity

The Handbike is steered by tilting the steering/crank column to the side. This movement is conveyed to the front wheel by means of a steering rod. Depending on the adjustment, the same angle of tilt can produce a sharper or weaker angle of front-wheel turn. The steering sensitivity is adjusted by fastening the steering rod in one of the three holes (1) on the steering rod bracket:

Upper hole:	produces a small angle of turn, large turning radius, subtle steering response
Middle hole:	produces a medium angle of turn, medium turning radius, moderate steering response
Lower hole:	produces a large angle of turn, small turning radius, sportier steering response

Note:

Be sure to use the upper hole until you have gained some experience and confidence with the tilt-steering system.

1. Unscrew the bolt (1) from the steering rod holder using a 6mm Allen key and remove it from the hole.
2. Insert the bolt (1) through the desired hole; then screw it tightly into the steering rod holder.

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Riding your Handbike

Mounting and dismounting

The steering/crank column can be folded forward to allow for easy mounting and dismounting.

1. Compress the left brake lever and wrap the Velcro strap around the handle to engage the parking brake and prevent the Handbike from rolling.
2. Pull up the locking mechanism (1) of the support bar and disengage the bar from the ball-head mounting bracket. Raise the steering/crank column to make additional space for mounting or dismounting.
3. After you have mounted or dismounted the trike, lower the steering/crank column and lock it back into place (reengage locking mechanism onto ball-head bracket).

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Shifting gears with the twist shifter

Turn the twist shifter (1) on the right crank handle clockwise to shift to a higher gear for faster riding or counterclockwise to shift to a lower gear for slower riding.

When riding uphill, you can also engage the Mountain Drive gear system.

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Shifting gears with the Mountain Drive gear system

A gear system with gear reduction is built into the crank assembly of the Handbike, making it easier to ride uphill. By pressing a button, you can switch to a lower gear ratio, “mountain gear”, and back again. This button can be pressed to engage the system at any time, even when the rider is not pedaling or the trike is stopped.

1. Press the button on the left side of the crankset to shift into “mountain gear” (gear reduction).
2. Press the button on the right side (2) of the crankset to shift back into “normal gear”.

With the Mountain Drive system, the number of gears is doubled.

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Using the Handbike as a tandem

For tandem cycling, the Handbike can be hitched to a Kettwiesel (or vice versa) using a universal coupling (accessory) for tandem cycling. The Handbike can be used either as the towed vehicle or the towing vehicle, depending on which vehicle has been equipped with the universal coupling.

The front wheel of the towed vehicle is removed, and its fork mounted onto the holding fixture of the universal coupling on the towing vehicle.

If the Handbike is being towed, the rider of the towed Handbike must never countersteer. Countersteering in this situation could damage the steering assembly.